## CareStart™ COVID-19 Antigen Home Test

## USER INSTRUCTIONS

You must follow the test directions carefully to get an accurate result. Visit [accessbio.net](http://accessbio.net) to obtain the complete instructions for use.

**FOR USE UNDER EMERGENCY USE AUTHORIZATION (EUA) ONLY.**

**IMPORTANT: Swabbing the nostrils is critical for obtaining an accurate result. If you do not swab your nose, the device will produce a false negative result.**

**1** Wash your hands thoroughly for at least 20 seconds before the test.

**2** Unpack the test components from the tray.

**3** Remove the test cassette and place it on a flat, clean surface.

**4** Locate the extraction vial and gently peel off the aluminum foil seal, being sure to keep the vial upright and place it in the packaging tray.

PEEL OFF

**5** Locate a nasal swab and remove from the pouch. Be careful not to touch the swab tip.

**6** Gently insert the swab no more than 3/4 inch into the **LEFT** nostril. Then, slowly rotate the swab at least **5 times** in a circular path for a total of **15 seconds**. If you have questions, see the CDC Guidelines.

X5 15 in SEC

**7** Gently remove the swab from the **LEFT** nostril and place directly into the **RIGHT** nostril, repeating the process of rotating at least **5 times** in a circular path for a total of **15 seconds**. Remove the swab from the RIGHT nostril.

X5 15 in SEC

**8** Place the swab into the extraction vial. Rotate the swab vigorously at least **5 times**.

X5

**9** Remove the swab by rotating against the extraction vial while squeezing the sides of the vial to release the liquid from the swab. Discard the swab in trash.

**10** Close the vial by pushing the cap firmly onto the vial.

PUSH FIRMLY

**11** With your finger, mix thoroughly by flicking the bottom of the vial.

TAP

**12** Invert the extraction vial and hold the sample vertically above the sample well. Squeeze the vial gently. Allow **THREE (3)** drops of sample to fall into the sample well.

3X 90°

**13 Start a timer.** Read the result at **10 minutes**. The test result should not be read after 15 minutes.

**10 mins**

**IMPORTANT** Do not move or lift the test cassette during this time. Do not exit the mobile app during this process.

**Disposal** Dispose of all used test kit components and swab samples in household trash.

**Using Mobile Application** [Scan the QR code to download](#)

- Ensure you have an internet connection and download the App prior to starting the test
- Ensure you are using a compatible smartphone. (For a list of compatible smartphone OS systems, visit [www.accessbio.net/app](http://www.accessbio.net/app))
- Only open the foil pouch packaging of test cassette when the App instructed to do so.

Please start the test and follow the in-app self-paced, step-by-step test instructions.

- Download and open App, On/Go™ Mobile Application
- Download the App on the App Store or Google Play Store. Ensure you are connected to the internet during your test.
- Answer a few questions in the App
- Watch the instructional video.
- Follow step-by-step instructions for your test.

The App will assist in your visual result interpretation. Please follow the instructions provided in the App. You will be required to take a picture of the test device, and then look at the test cassette and answer questions about the result interpretation.

## Results Interpretation

Make sure you wait the full 10 minutes.

You will be able to interpret your test results by following the in-app interpretation instructions or those provided below.

**NOTE:** The test results should be read by visual and interpreted at 10 minutes after the sample application and interpretation of the results should not exceed 15 minutes as it may yield inaccurate results.

**+** COVID-19 Detected (Positive)

One purple-colored line next to "C" and one blue-colored line next to "T" indicates COVID-19 positive result.

**IMPORTANT** Look very closely! The color intensity in the test region will vary. Any faint colored line in the test region should be considered as positive.

A positive test result indicates that antigens from SARS-CoV-2 were detected, and the patient is very likely to be infected with the virus and presumed to be contagious. Test results should always be considered in the context of clinical observations and epidemiological data in making a final diagnosis and patient management decisions. You should self-isolate at home and avoid contact with others as per CDC recommendations to avoid spreading the virus to others.

**-** COVID-19 Not Detected (Negative)

One purple-colored line only next to "C" indicates a negative result.

**Re-test in 24-48 hours if your first test result is negative.**

A negative test result indicates that antigens from SARS-CoV-2 were not detected from the specimen. However, a negative result does not rule out COVID-19 and should not be used as the sole basis for treatment or patient management decisions, including infection control decisions. Negative results should be considered in the context of an individual's recent exposures, history, and the presence of clinical signs and symptoms consistent with COVID-19 and confirmed with a molecular assay, if necessary, for patient management.

**?** Invalid

Invalid barcode or absence of a purple-colored line next to "C".

Re-test with a COVID-19 test may be needed. An invalid test result indicates that your test has experienced an error and is unable to interpret the result of the test. You will need to re-test with a new test or consult a healthcare professional. If you still have symptoms, you should self-isolate at home and avoid contact with others prior to the retest.

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